



ALLIED MEDICAL LIMITED

The Eldor Disposable Infusion Pump

Directions for Use



ISO 9001 & ISO 13485

Indications for Use

- The Eldor pump is indicated for delivery of medications
 - Percutaneous – Around surgical sites
 - Intravenously
 - Via Central vascular access
- The Eldor pump is supplied sterile non pyrogenic
- The Eldor pump is for single use only
- Store out of direct sunlight

Cautions

- Do not use if package is damaged or open
- Do not resterilize or exceed maximum fill volume of pump
- The Eldor pump can exhibit a variance in accuracy of approximately + or - 10 %
- Nominal infusion rate is labeled on the inside of each device
- Infusion time can be influenced by the viscosity of medication concentration
- Use strict aseptic technique when filling device

ALLIED MEDICAL LIMITED ABN: 99 089 052 479
LEVEL 2, 87 ADELAIDE TERRACE, EAST PERTH 6004
PO BOX 6915, EAST PERTH 6892

PH: +61 8 9266 0190


FAX: +61 8 9226 1999

info@alliedmedical.com.au

www.alliedmedical.com.au

Filling the Eldor Infusion pump

1. Remove device from packaging
2. Remove white cap (A) from filling port on top of device and keep
3. Slip the one way filter that is supplied securely into refill port
4. Inject the desired volume of fluid via a syringe into device via one way filter. The elastomeric balloon will expand as solution is injected
5. Stand device will refill port at top, remove one way filter and expel any air that may have entered the balloon
6. When balloon is full as per the required volume and free of air, securely replace white cap on top of refill port
7. **To Prime the patient line** - remove cap from end of patient line and wait until first drop of solution appears
8. Attach medication label and connect to patient

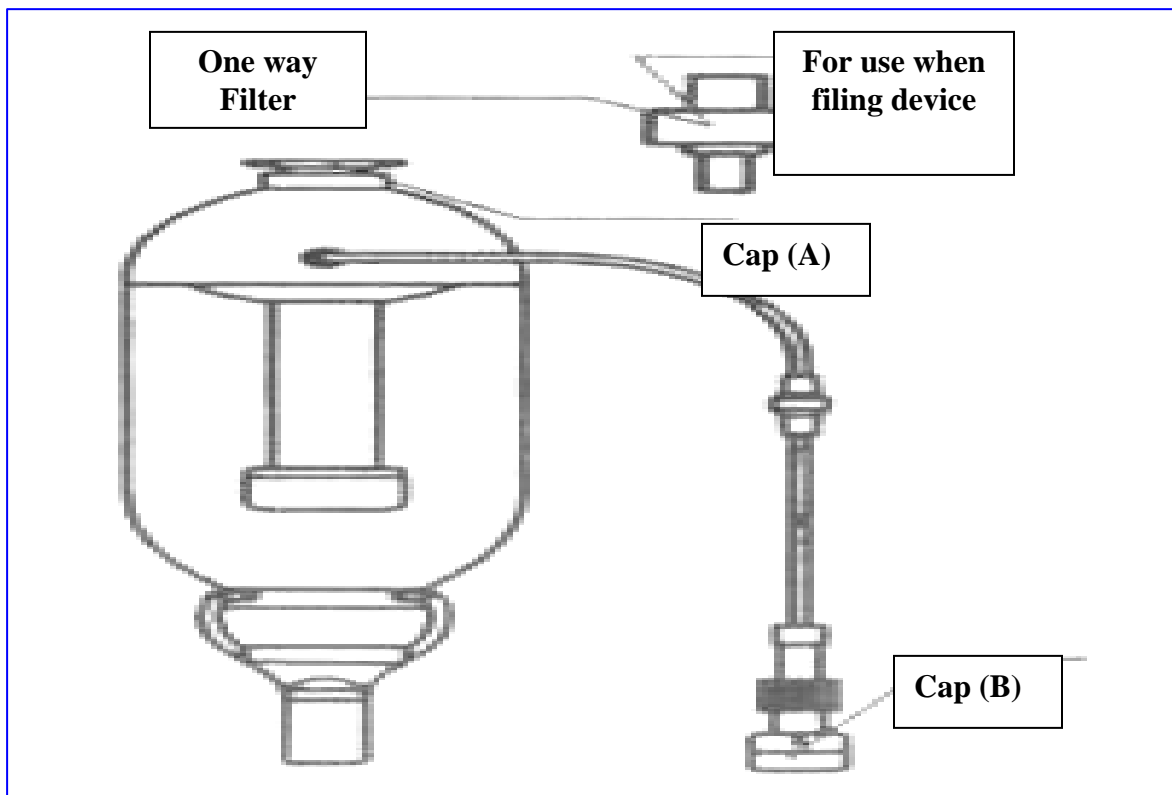

ALLIED MEDICAL LIMITED ABN: 99 089 052 479
LEVEL 2, 87 ADELAIDE TERRACE, EAST PERTH 6004
PO BOX 6915, EAST PERTH 6892

PH: +61 8 9266 0190

FAX: +61 8 9226 1999

info@alliedmedical.com.au

www.alliedmedical.com.au